Health Care Particulate Respirator and Surgical Mask

Description: The 3M Health Care Particulate Respirator and Surgical Mask 1870 is designed to help provide respiratory protection product has a filter efficiency level of 95% or greater against particulate aerosols free of oil. It is fluid resistant, disposable and can fit a wide range of face sizes.

This product contains no components made from natural rubber latex.

Intended Use: This product meets CDC guidelines for *M. tuberculosis* exposure control. As a respirator, it is intended to help reduce wearer exposure to certain airborne particles in a size range of 0.1 to > 10.0 microns, including those generated by electrocautery, laser surgery, and other powered medical instruments. As a surgical mask, it is designed to be fluid resistant to splash and spatter of blood and other infectious materials; when worn properly and in combination with protective eyewear, it complies with the OSHA Bloodborne Pathogens Standard. It also provides > 99% BFE* against wearer generated microorganisms.

Contraindications: Not for industrial use. Not for use with beards or other facial hair or conditions that prevents a good seal between the face and the sealing edge of the respirator.

Warning: This product does not eliminate the risk of contracting any disease or infection. Change immediately if contaminated with blood or body fluids. OSHA has not set a permissible exposure level for airborne biohazards.

Misuse may result in sickness or death.

Use Instructions:

1. Before use for respiratory protection, a written respiratory protection program must be implemented meeting all the requirements of OSHA 29 CFR 1910.139 such as medical evaluation, training and fit testing. The 3M sweet or bitter qualitative fit test is recommended for this respirator. When used only as a surgical mask, fit testing is not required.
2. The 1870 may be used until damaged, breathing becomes difficult, or contaminated with blood or body fluids. Otherwise, it may be stored and reused according to the facility's infection control policy.
3. Discard after every use when used for surgical procedures.


Technical Information: Call 1-800-228-3957. In Canada, call 1-800-563-2921.

*Tested against a 0.3 micron particle (mass median aerodynamic diameter) per 42 CFR 84.

*Bacterial Filtration Efficiency (BFE) determined by the modified Greene and Vesley test method.

Fitting Instructions: to be followed each time respirator is worn.

1. Remove respirator from packaging and hold with straps facing upward (1).
2. Fully open top and bottom panels, bending nosepiece around thumb at center of foam. Straps should separate when panels are opened. Make certain bottom panel is unfolded and completely opened (2).
3. Place respirator on your face so that the foam rests on your nose and the bottom panel is open under chin. Pull the top strap over your head and position it high on the back of the head (3). Then pull the bottom strap over your head and position it around the neck and below the ears. Adjust for a comfortable fit by pulling top panel toward the bridge of the nose and bottom panel under chin.
4. Place your fingertips from both hands at the top of the metal nosepiece. Using two hands, mold the nose area to the shape of your nose by pushing inward while moving your fingertips down both sides of the nosepiece (4).
 ⚠ Pinching the nosepiece using one hand may result in improper fit and less effective respirator performance. (Use two hands.)

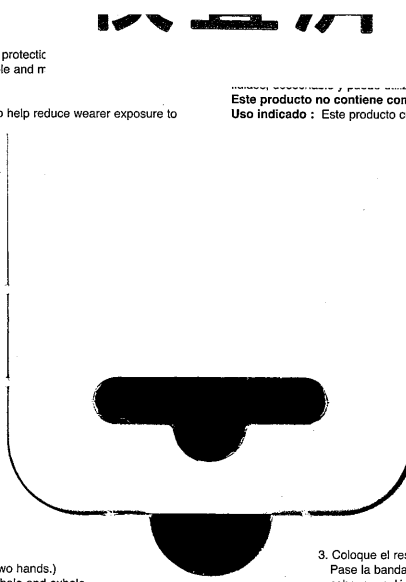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